


# FIELD MARKER


The radiotherapy Field Marker is a transfer seal type marker that assists to make marks precisely and conveniently during radiotherapy without fouling clothes, etc.

## Features


- ① Simple way of drawing a straight line precisely.
- ② Produces clear and fine lines without blotting.
- ③ No worry of fouling hands, clothes, etc. of surgeon or patient.
- ④ Long life durability.
- ⑤ Five ink colors for differentiation of irradiation sections or periods of application.
- ⑥ The special color product looks nearly transparent, unobtrusive, and fits for use at exposed sections.  
(It develops color under black light.) \*Black light is required.
- ⑦ It can be removed and readjusted easily if immediately after application.

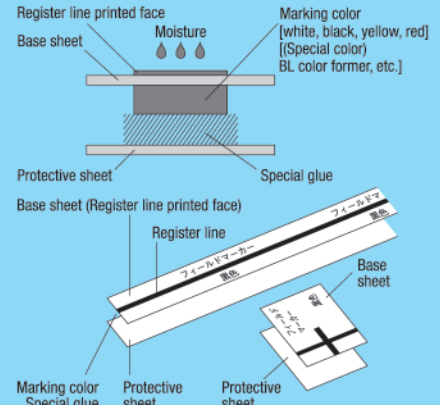


## Application

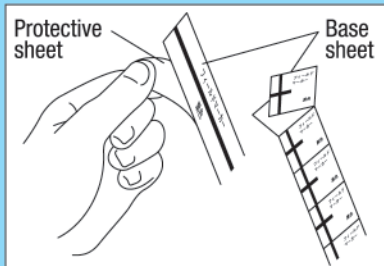

- ① Cut the Field Marker at a desired length.
- ② Remove the Field Marker applicator. (protective sheet)

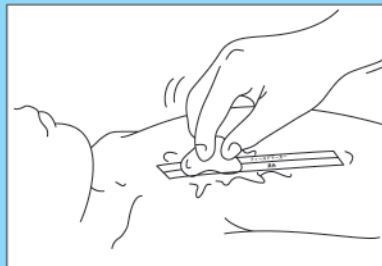

- ③ Attach the Field Marker on the skin along a line in the irradiation field.  
(Align the black line printed on the back of base sheet.)
- ④ Wet the surface of base sheet with ample amount of water.

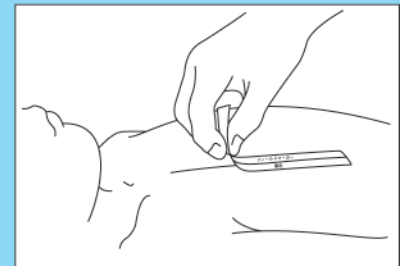

- ⑤ Remove the base sheet of Field Marker.

## Tips

- Clean oils from the skin to prolong the adhesion.
- Remove excessive water after wetting abundantly to prevent misalignment.
- Wait a few seconds after wetting abundantly before removing the base sheet successfully.
- Avoid rubbing the Field Marker after attachment for about half a day. If it is rubbed with clothes, etc. too earlier (within few minutes), it may come off.

## Type and color

Straight line : L = 200 (white, black, yellow, red and special color for development under black light),  
L = 400, L = 20 (each/white, black, yellow, red and special color for development under black light)  
Symbol : L (white, black, yellow, red and special color for development under black light)  
+, T (each/white, black, yellow and special color for development under black light)  
Circle + (black, red)

Note that we reserve the right to after specifications herein without notice for the purpose of improving our product.

## Cautions for use

1. When eruption, itches, redness or such after the use of the product, stop the use immediately and receive appropriate treatment.
2. Protect from direct exposure to sunlight, and store at a dry, cool place.
3. Avoid attaching the line section to a bruise, etc.
4. This is a non-sterilized product.
5. Do not sterilize.
6. Do not use at an antiseptic area.
7. For external application only.
8. If you may have any question, please contact the following.

## Agency

### Orion Electric Co., Ltd.

5-8-24, Sakae, Naka-ku, Nagoya-shi 460-0008, JAPAN  
Phone : +81-52-261-0706 Facsimile : +81-52-261-0798

## Distributed by

### Chubu Medical Co., Ltd.

33-5, Matsubara-cho, Yokkaichi-shi, Mie 510-8015, JAPAN